



Laundry365 Medical


We specialise in sluice room infection control solutions

Laundry365 has over 20 years experience in supplying and installing infection control solutions to the care sector.

Our range of products are designed to help care homes and hospitals to improve process and reduce the risk of infection.

We can supply a full range of washer/disinfectors, sinks, handwash basins, utensil racks and utensils.

Below is a typical sluice room layout.

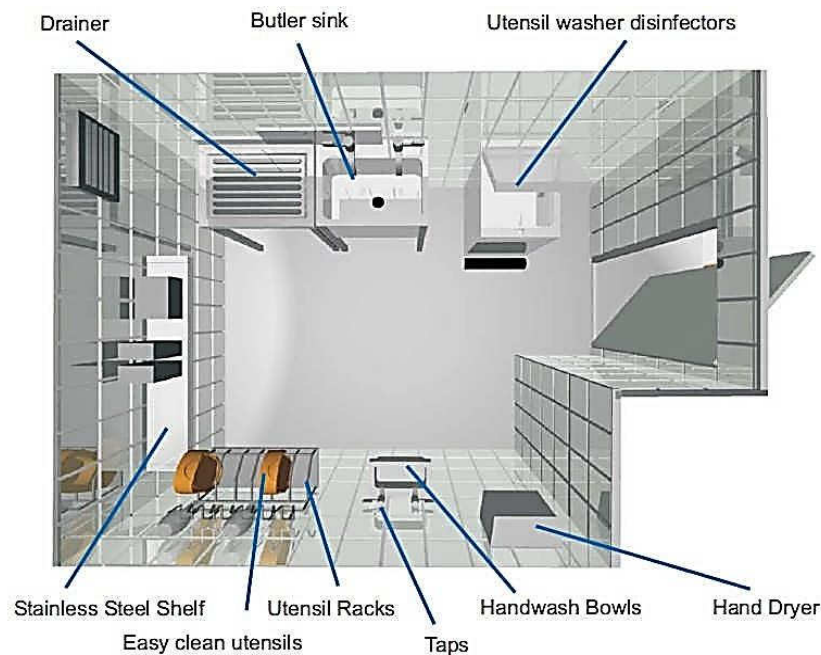



The CS1 ST WASHER / DISINFECTOR



The Compact Superior 1 washes and disinfects utensils one at a time and is specially designed for the small nursing home where space is limited.

The CS1 provides an effective infection control barrier which protects residents and staff from the risk of cross infection and management from potential liabilities. All Laundry365 washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance

Laundry365 utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size
Height
Width
Depth

Installation
910mm (36")
435mm (18")
660mm (26")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at 80°C for 70 seconds -
Rinse to cool

2. Super wash
Pre flush - Wash -
Disinfection at 80°C for 70 seconds - Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

Waste

82mm (3") 'P' or 'S' trap





The CS2 ST WASHER / DISINFECTOR



The Compact Superior 2 is specifically designed for multi-storey care homes where code of practice requires a sluice room on each floor

The CS2 medium sized top loading machine washes and disinfects a variety of utensils two at a time. For care homes committed to disposable products it is an ideal, versatile washer/disinfector for the carriers and reusable utensils to complete the infection control barrier. All Laundry365 washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance.

Laundry365 utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings.

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size
Height
Width
Depth

Installation
910mm (36")
435mm (18")
660mm (26")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at
80°C for 70 seconds -
Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

2. Super wash
Pre flush - Wash -
Disinfection at 80°C for 70
seconds - Rinse to cool

Waste

82mm (3") 'P' or 'S' trap





The CS3 ST WASHER / DISINFECTOR



The Compact Superior 3 ST Commode Pot & Bottle machine is designed for the full bed occupancy and high dependency nursing homes which have peak loads and a high utensil volume.

The CS3 ST has the capacity to efficiently wash and disinfect 3 utensils in one operation, cutting water and heating costs and staff waiting time and utensil queuing. All Laundry365 washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance.

Laundry365 utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply



Size	Installation	Electrical	230V 50Hz 16A single phase
Height	975mm (39")		Fuse - mains 16A - control 5A
Width	600mm (24")		
Depth	710mm (28")		
Water Feed		Washing	1. Normal wash
Supply	Cold	cycles	Wash - Disinfection at 80°C for 70
Connection	1/2" BSP male parallel		seconds - Rinse to cool
Overflow	1x19mm (3/4") bore. End must be visible. (UK requirement)		2. Super Wash
			Pre flush - Wash - Disinfection
			at 80°C for 70 seconds -
			Rinse to cool
Waste	82mm (3") 'P' or 'S' trap		





The CS3 HD WASHER / DISINFECTOR




The Compact Superior 3 HD **multi-utensil** machine is designed for the full bed occupancy and high dependency nursing homes which have peak loads and a high utensil volume.

The CS3 HD has the capacity to efficiently wash and disinfect 3 utensils in one operation, cutting water and heating costs, staff waiting time and utensil queuing.

All Laundry365 washer/disinfectors can include the DEOSCALE water treatment system to control limescale build-up and reduce maintenance. Laundry 365 utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings

WRAS Approval No. 0310064 Internal break tank has a type 'A' air gap supply

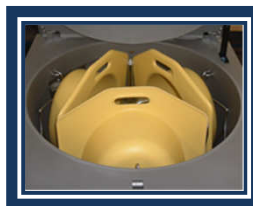

Size
Height
Width
Depth

Installation
1100mm (44")
600mm (24")
710mm (28")

Electrical

230V 50Hz 16A single phase
Fuse - mains 16A - control 5A

Water Feed
Supply
Connection

Cold Feed x 2
1/2" BSP male parallel

Washing
cycles

1. Normal wash
Wash - Disinfection at 80°C for 70 seconds - Rinse to cool

Overflow

1x19mm (3/4") bore.
End must be visible.
(UK requirement)

2. Super Wash
Pre flush - Wash - Disinfection at 80°C for 70 seconds - Rinse to cool

Waste

82mm (3") 'P' or 'S' trap





SHELVING DRAINERS & WORK SURFACES



Vogue Stainless Steel Midi Sink

Manufactured from 304 grade stainless steel
600mm deep unit with a raised lip to prevent spillage and things slipping off.

Adjustable feet and strong undershelf.

Dimensions HxWxD - 960mm x 750mm x 600mm



Vogue Mixer Taps

WRAS Approved

Hot and Cold Levers

90 degree turn

Finish - Polished Stainless Steel

Dimensions HxWxD - 223mm x 212mm x 195mm



Vogue Knee Operated Sink

Manufactured from 304 grade stainless steel

Complete with drain boss, pre mixer valve and spout

Bowl Size HxWxL - 120mm x 345mm x 245mm

Dimensions HxWxD - 185mm x 440mm x 330mm



Heavy duty wall mounted shelves

With a standard width of 300mm with integral wall fixing brackets.

Shelving 600

Shelving 1200

Shelving 1500

Shelving 1800

Shelving 2000



POTS - PANS & BOTTLES



Commode Insert pan

The insert commode pan improves the commode chair use, as well as sample gathering in the toilet suite.



Slipper pan

The slipper pan has no end pocket to trap soil and eliminates cleaning problems. Three hand grips allow easy positioning. An indent on the plateau reduces point load on the spine. Smooth movement over the mattress is assured.



In-Bed pan

The in-bed pan gives confidence, stability and secure feeling for sitting patients. The buttress edge gives extra support on the mattress, especially backwards. A raised edge helps location and comfort.



Urine bottle


Robust opaque polythene urine bottle, graduated in ml to 1 litre. Complete with snap lid for transit safety.



PAN BOTTLE & UTENSIL RACK

The Amethyst Economic (non-heated) quality stainless steel racks are designed to store bedpans, commode pots and urine bottles efficiently and safely.

All racks are manufactured to an exceptionally high standard in electro polished grade 304 stainless steel with a removable stainless drip tray for easy cleaning and disinfection.

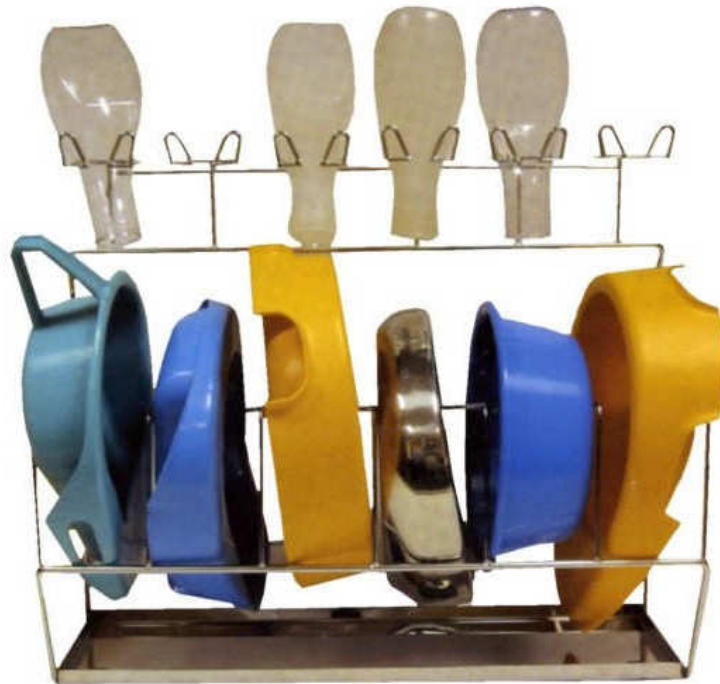

M
e
d
i
c
a
l



Head Office

Unit 426
37 St Andrews Street
Norwich
Norfolk
NR2 4TP

Scottish Office

Block A Unit 1
Smeaton Road
Dundee
DD2 4UT

Tel: 0800 0699 365
Email: info@laundry365.co.uk Web:
www.laundry365.co.uk

